



HEPATITIS C VIRUS RAPID TEST

Wondfo One Step Hepatitis C Virus Whole Blood/Serum/Plasma Test Strip is a rapid chromatographic immunoassay for the qualitative detection of antibody to Hepatitis C Virus in whole blood/serum/ plasma. For in vitro diagnostic use only. For healthcare professional use only.

General Information:


- Sensitivity : 99%, Specificity : 99.8%
- Specimen : Blood, Serum or Plasma
- Reading Time : 15 min
- Storage : 2~30°C
- 25 tests / kit



Procedures:


For whole blood testing:

1. Holding a sample dropper vertically, add two drops of whole blood (about 50 μ l) into the sample pad behind the arrow mark at the bottom of test strip.
2. Add two drop (about 50 μ l) of dilution buffer (provided) from the bottle directly into the same pad.

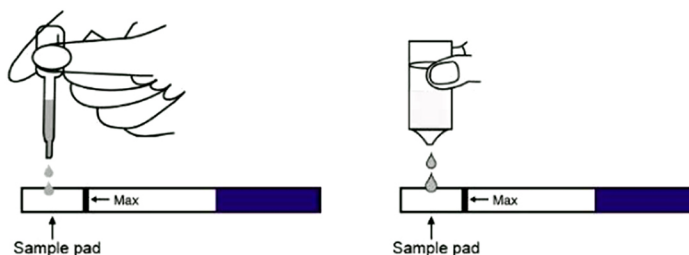

For serum / plasma testing:

1. Holding a sample dropper vertically, add three drops (60 μ l to 80 μ l) of serum or plasma into the sample pad.

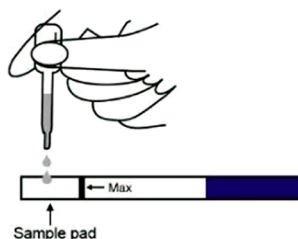

Read the results:

